


Approved by:	Version: 20230906	
Date:	Author: FW	Page 1 by 1

## Alere LDX Cholestech Instrument-tag

The document describes the connection and labeling of the meter, to work with the Zafena POC-Workstation. Physical connection is between RJ11 and USB-A.

Cable product code:

**Cholestech connect cable RJ11 to DB9m + ZAF-523**

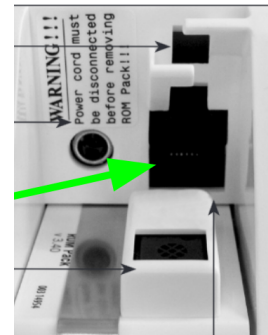

## Instrument-tag: LMC-ACL1

Print and stick the instrument tag on to or next to the instrument. Follow the documentation "Quick guide" to perform analysis and submit results.



## Communication settings

Printer option must be set to: **ON**

Press **DATA** to select **PRINTER OPTIONS**:

–**On** (Printer function turned on.) or

–**Off** (Printer function turned off.)

Press **RUN** to continue