

Approved by:	Version: 20230907
Date:	Author: FW

SpotChem EL SE-1520

Instructions to connect to Zafena POC-Workstation

The document describes the connection and labeling of the meter, to work with the Zafena POC-Workstation. Physical connection is between RJ11 and USB-A.

Cable product code:

1x **ZAF-520-13**

1x **ZAF-520-1**

Instrument-tag: LMC-ASE2

Print and stick the instrument tag on to or next to the instrument. Follow the documentation "Quick guide" to perform analysis and submit results.



LMC-ASE2

Procedure

1. Attach the cable ZAF-520-13 to the meter. On the back of the meter there is a socket where the cable connector fits. When connecting, you should hear a click when you insert the cable all the way.
2. Connect ZAF-520-13 serial end to ZAF-520-1 and the USB into one of the USB-ports on the POC-Workstation.
3. To test the connection:
 - A. Scan the instrument-tag on the meter.
 - B. Through the meter menu, choose: "Submenu" (press **2**), then "Result" (press **1**), then "Send" (press **2**), then "Latest" (Press **1**).



The transfer starts and the results are presented on the POC Workstation.