

Approved by:	Version: 20210915	
Date:	Author: FW	Page 1 by 1

SpotChem EZ SP-4430

Instructions to connect to Zafena POC-Workstation

The document describes the connection and labeling of the meter, to work with the Zafena POC-Workstation. Physical connection is between RJ11 and USB-A.

Cable product code:

1x **ZAF-520-13**

1x **ZAF-520-1**

Instrument-tag: LMC-ASE1

Print and stick the instrument tag on to or next to the instrument. Follow the documentation "Quick guide" to perform analysis and submit results.



Procedure

1. Attach the cable ZAF-520-13 to the meter. On the back of the meter there is a socket where the cable connector fits. When connecting, you should hear a click when you insert the cable all the way.
2. Connect ZAF-520-13 serial end to ZAF-520-1 and the USB into one of the USB-ports on the POC-Workstation.
3. To test the connection:
 - A. Scan the instrument-tag on the meter.
 - B. Through the meter menu, choose: "Submenu" (press **2**), then "Result" (press **1**), then "Send" (press **2**), then "Latest" (Press **1**).



The transfer starts and the results are presented on the POC Workstation.