

Approved by:	Version: 20230901
Date:	Author: FW Page 1 by 2

Skyla VB1 Instructions

The document describes the connection and labeling of the meter, to work with the Zafena POC-Workstation. Physical connection is between USB-A and RS232.

Product code cable: **ZAF-523**




Instrument-tag: LMC-SVB1

Print and attach the instrument-tag on to the meter. In some cases, to enhance the work flow, it might work better to attach the tag in close proximity to the meter.



Communication settings

1. Choose "Settings" and then "Administrator settings" (universal password 5787722)

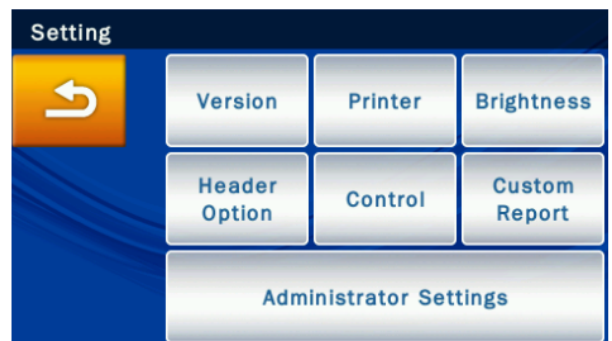

2. Choose "RS232 Baud Rate"



Approved by:	Version: 20230901
Date:	Author: FW Page 2 by 2

3. Set Baud Rate to:
'115200'



4. Press "OK" and then the "House" to get back to the home screen.
5. Follow the quick guide to start your digitalization.